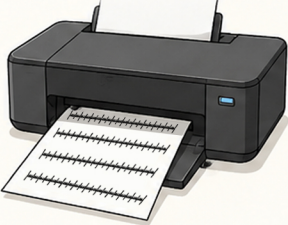
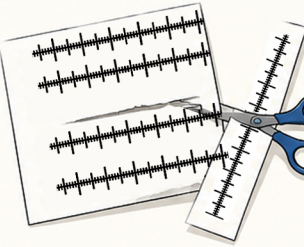
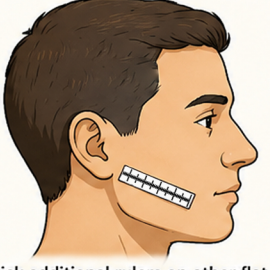
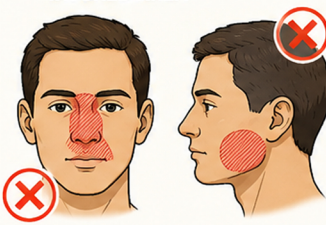
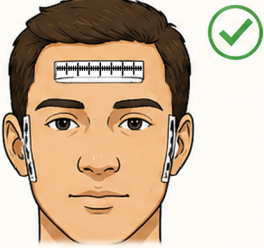


## HOW TO USE THE SCALE RULER FOR 3D FACE SCAN

This ruler helps to bring your 3D face scan to real world scale in 3D software.

|                                                                                                                                                                                                                                                               |                                                                                                                                                                                                                                          |                                                                                                                                                                                                                                  |                                                                                                                                                                                                                                       |
|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|---------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|
| <p><b>1 PRINT THE RULER</b></p>  <p>Print this PDF at 100% (A4 Paper).<br/>Do not fit to page or scale to margins.</p>                                                       | <p><b>2 CUT THE RULERS</b></p>  <p>Cut along the dotted lines and separate the individual rulers.</p>                                                   | <p><b>3 STICK ON FOREHEAD (FLAT AREA)</b></p>  <p>Stick one ruler on the forehead in the flattest possible area.</p>                           | <p><b>4 STICK ON OTHER FLAT AREAS</b></p>  <p>Stick additional rulers on other flat areas like temple, side of head, cheek, or under the chin.</p> |
| <p><b>5 AVOID AREA OF INTEREST / INJURED AREA</b></p>  <p><b>Do NOT</b> stick rulers on or near the area of interest or injured area. Keep them away from these regions.</p> | <p><b>6 CHECK ADHESION &amp; VISIBILITY</b></p>  <p>Make sure all rulers are firmly stuck, clearly visible and not overlapping important features.</p> | <p><b>7 CAPTURE 3D SCAN</b></p>  <p>Capture your 3D face scan as usual. The rulers will be used as reference to set the real-world scale.</p> |                                                                                                                                                                                                                                       |

 **TIPS:** Use as many rulers as needed on different flat areas for better accuracy. | Ensure good lighting and the rulers are not reflecting light. |  Your scan is now to scale!

- Print at 100% / A4 Paper (Do not resize)
- Cut the rulers — 60 mm (6 cm) or longer is recommended for better accuracy (smaller pieces can also be used)

- Stick on flat areas — forehead (best), cheeks, or side of face
- Do not place on injured / treatment area
- Stick rulers flat and firmly — no folds or lifting
- Use as many rulers as possible for better accuracy

